




ADHESIVE KNIT



Sport SKU



Medical SKU

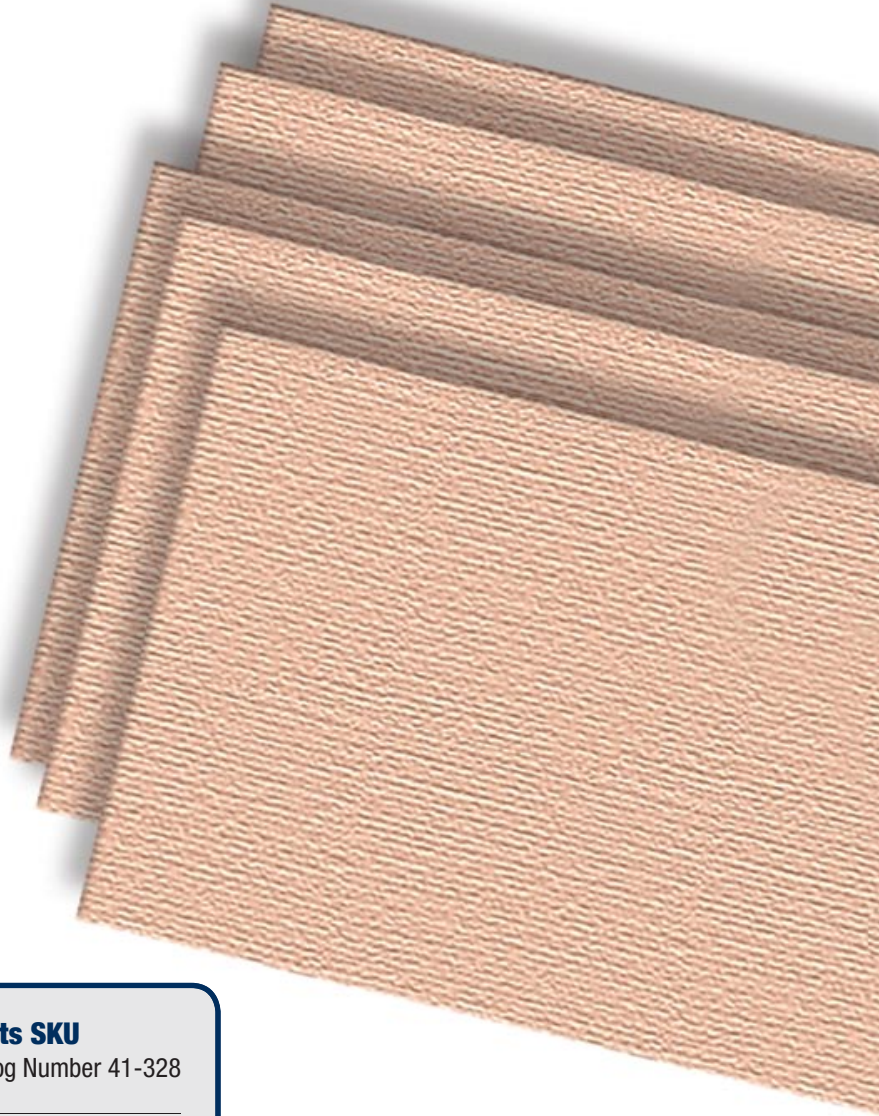

Sports SKU

Catalog Number 41-328

Medical SKU

Catalog Number 41-301





ADHESIVE KNIT

TARGET USER:

General users on uncompromised skin for prevention of skin pressure, rubbing, chafing, and blisters.

RECOMMENDED FOR:

Prevention of pressure, rubbing, chafing, and blisters. Ideal for joints and other areas where movement occurs.

COMPONENTS:

6 – 3" x 5" adhesive knit sheets

Adhesive Knit can be used anywhere on the body where rubbing, chafing or friction occur. Cut to any size. Adhesive Knit is made from a highly breathable, medical grade, non-woven fabric with the ability to stretch, conform, and fit securely against the skin. It is ideal to hold 2nd Skin® Moist Pads in place.

FEATURES & BENEFITS:

- Medical grade adhesive
- Flexible
- Non-irritant

INSTRUCTIONS:

1. Cut Adhesive Knit to desired size and shape.
2. Make sure skin is dry and grease-free before applying.
3. Remove backing and apply to skin.
4. After use, alcohol or baby oil may be used to dissolve adhesive residue.



SPORTS SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
41-328	3" x 5"	6	1" x 3 3/4" x 6 1/2"	11 3/8" x 8 1/4" x 6 5/8"	24

MEDICAL SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
41-301	3" x 5"	6	1" x 3 1/2" x 6 1/16"	13" x 14 9/16" x 7 1/16"	48

©2011 Spenco Medical Corporation. All Rights Reserved.
® Registered and ™ Trademark of Spenco Medical Corporation



2nd Skin®

Spenco Medical Corporation
1-800-877-3626
www.spenco.com
In Canada call: 1-800-387-9538