



# AQUAHEAL® Hydrogel Bandages



**Sport SKU**



**Institutional SKU**



**Medical SKU**



### Sports SKU

Catalog Number 48-235    Large - 1 3/4" x 3"    (3 Large)  
Small - 1" x 2 1/5"    (3 Small)

### Institutional SKUs

Catalog Number 48-237    Large - 1 3/4" x 3"    (25 Pack)  
Catalog Number 48-236    Small - 1" x 2 1/5"    (25 Pack)

### Medical SKU

Catalog Number 48-239    Large - 1 3/4" x 3"    (3 Large)  
Small - 1" x 2 1/5"    (3 Small)



**2nd Skin®**



## AQUAHEAL® Hydrogel Bandages

### TARGET USER:

General users seeking a first aid solution to a variety of injuries of the skin.

### RECOMMENDED FOR:

Protection and management of minor cuts, scrapes, bites, blisters, and burns.

### COMPONENTS:

Sterile Hydrogel with polyurethane film adhesive

### FEATURES & BENEFITS:

- Transparent – allows user to see the wound's healing progress
- Provides Instant Relief
- Promotes Natural Healing
- Soothing and Cooling
- Seals Bacteria Out
- Breathable
- Protects and Cushions
- Highly Absorbent
- Multi-Day Use

### SPORTS SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
48-235	Large - 1 3/4" x 3"	3 Large	1" x 3 3/4" x 6 1/2"	11 3/8" x 8 1/4" x 6 5/8"	24
	Small - 1" x 2 1/5"	3 Small			

### INSTITUTIONAL SKUs

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
48-237	1 3/4" x 3"	25	3" x 4 3/16" x 2 1/8"	12 1/4" x 11 1/8" x 4 3/8"	20
48-236	1" x 2 1/5"	25	2 5/16" x 3 1/2" x 2 1/8"	11 1/4" x 9 3/8" x 3 3/4"	20

### MEDICAL SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
48-239	Large - 1 3/4" x 3"	3 Large	13/16" x 3 1/8" x 5 5/8"	9 15/16" x 5 1/2" x 3 3/8"	12
	Small - 1" x 2 1/5"	3 Small			

### INSTRUCTIONS:

Apply to skin that is clean and dry.

1. Remove white plastic liner.
2. Without touching the hydrogel, apply the bandage in rolling motion, with the hydrogel completely covering the wound, and press into place.
3. Peel off printed paper backing and smooth out the bandage.



**Caution: Do not use if pouch is damaged or opened. Do not use if product is dried out. Use only once. Use before expiration date marked on carton. If complications occur, consult a physician.**

©2011 Spenco Medical Corporation. All Rights Reserved.  
® and ™ Trademark of Spenco Medical Corporation



**2nd Skin®**

Spenco Medical Corporation  
1-800-877-3626  
www.spenco.com  
In Canada call: 1-800-387-9538