




BLISTER KIT



Sport SKU



Medical SKU

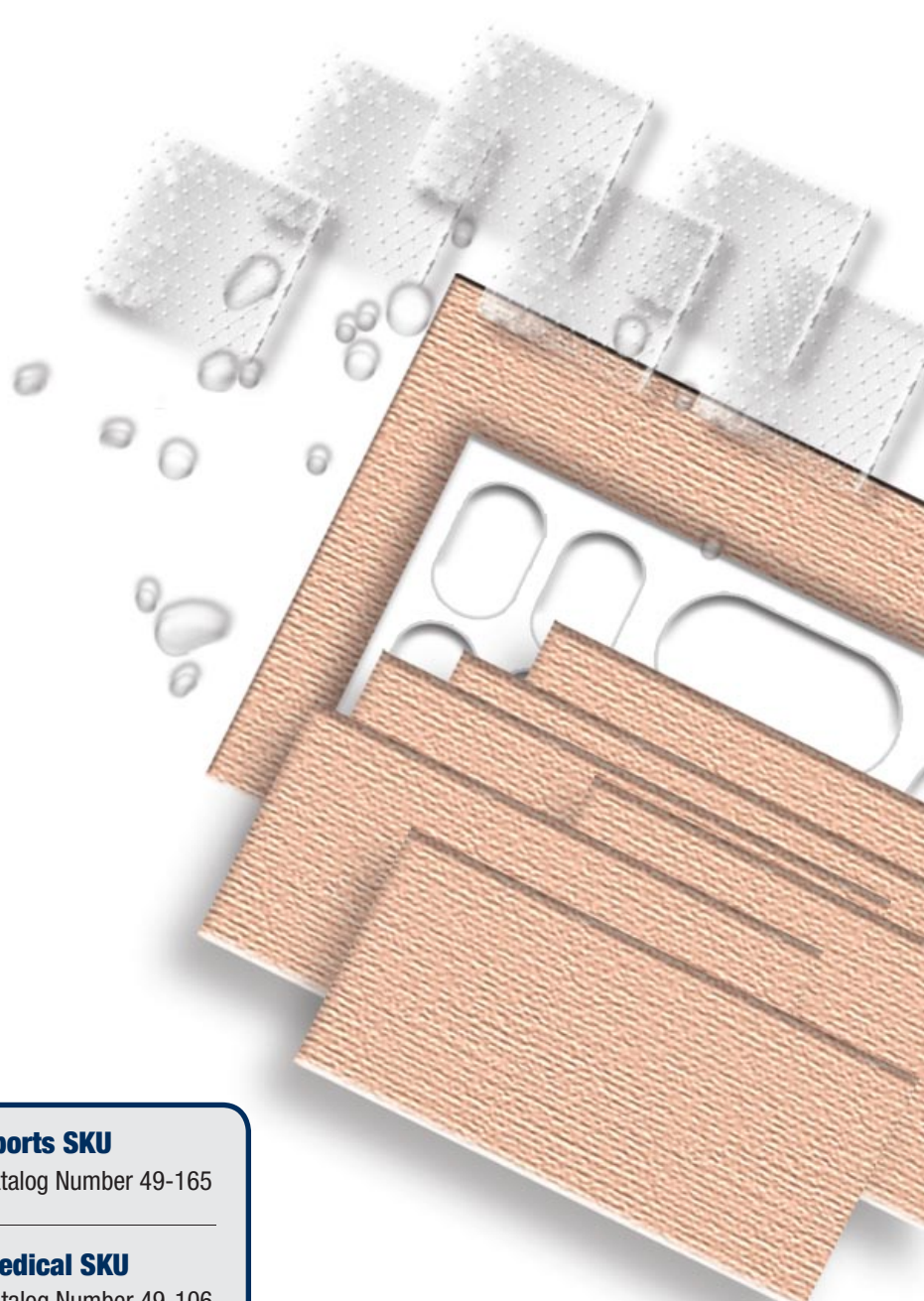

Sports SKU
 Catalog Number 49-165

Medical SKU
 Catalog Number 49-106





BLISTER KIT

TARGET USER:

General users seeking prevention of skin pressure, rubbing, chafing, and blisters on uncompromised skin.

RECOMMENDED FOR:

Prevention of skin pressure, rubbing, chafing, and blisters. Not to be used on open wounds.

COMPONENTS:

13 Assorted Dressings
11 Pressure Pads

2nd Skin® Moist Pads (non-sterile) are hydrogel pads made from 95% pure water. They provide a moist, comfortable dressing to help keep the skin clean.

Adhesive Knit is made from a highly breathable, medical grade, non-woven fabric with the ability to stretch, conform, and fit securely against the skin. It is ideal to hold 2nd Skin® Moist Pads in place.

Pressure Pads protect “hot spots” on the skin against rubbing and friction.

FEATURES & BENEFITS:

- Convenient protection and prevention
- Medical grade adhesive
- Flexible
- Soothing, cooling, and moisturizing
- Non-irritant
- Odorless
- Provides instant relief
- Reduces pain

INSTRUCTIONS:

1. Remove blue backing and apply 2nd Skin® Moist Pad directly on the blister-prone area.
2. Cover 2nd Skin® Moist Pad with Adhesive Knit.
3. Apply Pressure Pad to “hot spot” areas on the skin.
4. Always reseal unused portions of the 2nd Skin® Moist Pads to prevent from drying.



SPORTS SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
49-165	Assorted Sizes	13 Assorted Dressings 11 Pressure Pads	1" x 3 3/4" x 6 1/2"	11 3/8" x 8 1/4" x 6 5/8"	24

MEDICAL SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
49-106	Assorted Sizes	13 Assorted Dressings 11 Pressure Pads	1" x 3 1/2" x 6 1/16"	15" x 13 1/2" x 7 1/2"	48

©2011 Spenco Medical Corporation. All Rights Reserved.
® and ™ Trademark of Spenco Medical Corporation



2nd Skin®

Spenco Medical Corporation
1-800-877-3626
www.spenco.com
In Canada call: 1-800-387-9538