



# BLISTER PADS



**Sport SKU**



**Medical SKU**



**Sports SKU**  
 Catalog Number 47-424

---

**Medical SKU**  
 Catalog Number 47-423



**2nd Skin®**



## BLISTER PADS

### TARGET USER:

General users with compromised skin due to bruises and blisters as well as healthy skin for the prevention of blisters, chafing, and rubbing.

### RECOMMENDED FOR:

Prevention and treatment of blisters, bruises, and minor cuts.

### COMPONENTS:

5 - Sterile hydrocolloid pads

### FEATURES & BENEFITS:

- Waterproof
- Highly-conformable
- Stays up to five days
- Thin
- Provides a bacterial barrier

### INSTRUCTIONS:

1. Clean and dry the affected area.
2. Remove the large release liner from the product, exposing the adhesive side.
3. Without touching the center of the pad, apply to the affected area, and smooth it over using a rolling motion. Remove the small release liner.
4. When pad loosens itself over time, remove gently by pulling away from the skin. The pad should not be left on longer than five (5) days without changing.
5. Reapply a fresh pad and continue application until the blister is healed.
6. The product may also be used for cushioning to prevent blisters from occurring.

Single use only. Sterility guaranteed unless pouch is damaged or opened. If signs of infection occur (increased bleeding, pain, swelling) discontinue use and consult a physician.



### SPORTS SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
47-424	1 1/2" x 2 2/5"	5	1" x 3 3/4" x 6 1/2"	11 3/8" x 8 1/4" x 6 5/8"	24

### MEDICAL SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
47-423	1 1/2" x 2 2/5"	5	13/16" x 3 1/8" x 5 5/8"	9 13/16" x 5 1/2" x 3 3/8"	12

©2011 Spenco Medical Corporation. All Rights Reserved.  
® and ™ Trademark of Spenco Medical Corporation



**2nd Skin®**

Spenco Medical Corporation  
1-800-877-3626  
www.spenco.com  
In Canada call: 1-800-387-9538