

Scar Gel
Cat. No. 47-548



 **Target User:**

General users with hypertrophic or keloid (raised) scars.

 **Recommended For:**

Treating red and raised scars (old and new) that often occur following surgery, an accident or a burn. Not to be used on open wounds.

 **Components:**

15g Silicone-based gel

 **Instructions:**

- 1.) Clean and dry hands and the scar site
- 2.) Apply a thin coat on the affected area, allowing to air dry
- 3.) Upon reapplication, wash the scar site in mild soapy warm water and dry
- 4.) Follow this procedure daily. Recommended reapplication after bathing, exercising or as needed
- 5.) If a rash or itching occurs, clean the affected area with soapy water. If symptoms persist, discontinue use and consult your doctor.
- 6.) Store the gel in a cool, dry place when not in use.

Features & Benefits:

Can be worn during the day or overnight

Fast drying

Colorless

Flexible

Leaves a thin, smooth coating

Call toll-free: 1-800-877-3626
In Canada: 1-800-387-9538

 www.spenco.com

