



SCAR PADS



Institutional SKUs

Catalog Number 10-535	3" x 4"	(1 pad)
Catalog Number 10-536	3" x 4"	(10 pads)
Catalog Number 10-537	4 3/4" x 6"	(1 pad)



2nd Skin®



SCAR PADS

TARGET USER:

General users seeking non-invasive, self-treatment for raised and discolored scars (old and new).

RECOMMENDED FOR:

Designed to prevent or reduce the appearance of raised and discolored scars whether they are new or old. Scarring may result from 1st and 2nd degree burns, surgical incisions, scratches, cuts, acne and stretch marks.

COMPONENTS:

100% Silicone Gel Sheet

FEATURES & BENEFITS:

- Ideal for scars resulting from surgical incisions, scratches, cuts, acne scars and stretch marks
- Reusable
- Self-adhesive
- Semi-transparent
- Cut to size
- Washable

INSTRUCTIONS:

1. Remove protective liner from adhesive side of Scar Pad.
2. Apply to scar without stretching. In areas with greater movement, a light bandage or tape can give further security.
3. The pad should be used for at least 12 hours a day.
4. Clean the Scar Pad's adhesive side after use with a mild soap and rinse well with clean water.
5. Between applications the protective liner can be re-applied and the pad can be stored in the tray provided.

Caution: Do not use on open wounds. If a rash occurs, discontinue use and seek medical advice. Do not use after the expiration date. Store in cool, dry place.



INSTITUTIONAL SKUs

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
10-535	3" x 4"	1	13/16" x 3 5/8" x 4 7/8"	10 3/8" x 5 5/8" x 4 1/2"	12
10-536	3" x 4"	10	1 5/8" x 3 5/8" x 4 7/8"	10 5/8" x 10 3/8" x 4 1/2"	12
10-537	4 3/4" x 6"	1	13/16" x 5 5/8" x 7"	10 5/8" x 7 1/2" x 6 1/2"	12

©2011 Spenco Medical Corporation. All Rights Reserved.
® Registered and ™ Trademark of Spenco Medical Corporation



2nd Skin®

Spenco Medical Corporation
1-800-877-3626
www.spenco.com
In Canada call: 1-800-387-9538