



# HYDROGEL JARS

Institutional SKUs



### Institutional SKUs

Catalog Number 10-603	3" Circles	(48 Pack)
Catalog Number 10-637	1" Squares	(200 Pack)
Catalog Number 10-638	2" x 3" Rectangles	(48 Pack)





## HYDROGEL JARS

### TARGET USER:

General users on uncompromised skin for prevention of skin pressure, rubbing, chafing, and blisters.

### RECOMMENDED FOR:

Prevention of skin pressure, rubbing, chafing, and blisters. Jar format is ideal for use in clinics, training rooms and first aid stations.

### COMPONENTS:

Hydrogel pads made out of at 95% pure water.

Available in 3" circles, 1" squares, and 2" x 3" rectangles.

### FEATURES & BENEFITS:

- Soothing and cooling
- Non-irritant
- Odorless
- Provides instant relief
- Protects and cushions

### INSTRUCTIONS:

1. Cut hydrogel to desired size.
2. Remove blue backing and apply gel to the skin.
3. Secure hydrogel with adhesive tape.
4. If hydrogel dries on the skin, simply soak it with clean water. It can then be easily removed.
5. Change daily.



### SPORTS SKU

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
10-603	3" Circles	48	3 9/16" dia. x 3 3/16"	14 3/4" x 11" x 6 3/4"	24
10-637	1" Squares	200	3 9/16" dia. x 3 3/16"	14 3/4" x 11" x 6 3/4"	24
10-638	2" x 3" Rectangles	48	3 9/16" dia. x 3 3/16"	14 3/4" x 11" x 6 3/4"	24

©2011 Spenco Medical Corporation. All Rights Reserved.  
® Registered and ™ Trademark of Spenco Medical Corporation



## 2nd Skin®

Spenco Medical Corporation  
 1-800-877-3626  
 www.spenco.com  
 In Canada call: 1-800-387-9538